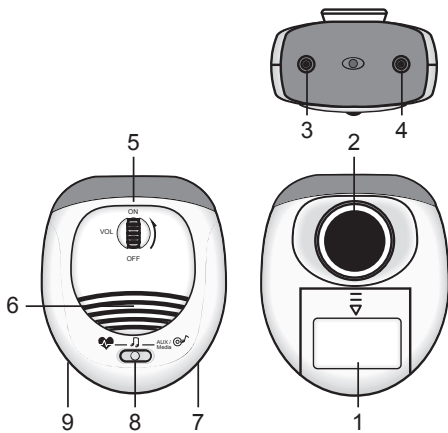
SWITEL

BH 170

Babysound-Monitor

DEUTSCH

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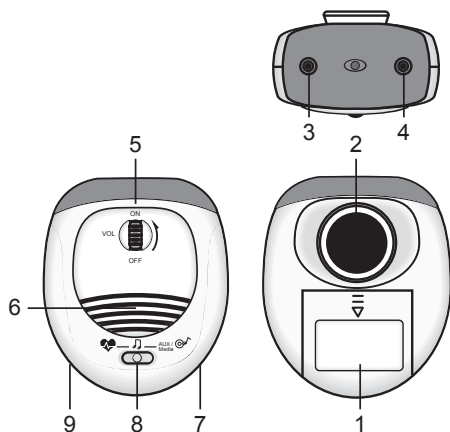
ENGLISH

Operating Instructions

Please read and observe the following information and keep the operating manual nearby for future reference!

WARNING!

Safety information

- ➔ Only use batteries of the same type!
- ➔ Always treat batteries with due care and attention and only use batteries as described.
- ➔ Never use standard and rechargeable batteries together. Never use rechargeable batteries or standard batteries with different capacities or charge statuses together. Never use standard or rechargeable batteries which are damaged.
- Risk of suffocation through small objects, packaging and protective foil!
- ➔ Keep children away from the product and its packaging!
- Risks to health and the environment from batteries!
- ➔ Never open, damage or swallow batteries or allow them to pollute the environment. They could contain toxic and ecologically harmful heavy metals.

ATTENTION!

- ➔ Prevent exposure to environmental influences, e.g. smoke, dust, vibration, chemicals, moisture, heat and direct sunlight.
- ➔ Only allow repairs to be completed by qualified personnel.

PACKAGE CONTENTS

1 BH 170 monitor, 1 microphone, 1 headphone, 1 audio cable, 1 Operating manual

OPERATING ELEMENTS

- 1 Battery cover
- 2 Listening cone
- 3 Headphone jack
- 4 Headphone jack
- 5 On/Off, Volume
- 6 Speaker
- 7 AUX
- 8 Function mode
- 9 Microphone jack

STARTING UP

- ➔ Install fresh alkaline „AA“ batteries.
After inserting the batteries, the BH 170 is ready for use.

Technical data (All rights reserved)

Feature	Value
BH 170 monitor	
W x H x D	97 x 116 x 57 mm
Weight	130 g (without battery)
Power	3 x 1,5V AA, not included

LISTENING

- ➔ Insert the Headphone plug into either jack (3 / 4) on the BH 170
- ➔ Place the „listening cone“ (2) to stomach or lower back
- ➔ Slide the „function mode button“ (8) to „♥“ position for listening to the womb sound.
- ➔ Turn on the unit and adjust the volume.
(Start with a low to moderate setting)

„MUSIC BOX“ PLAYING

- ➔ Slide the „function mode button“ (8) to „♪“ position, 5 different melodies will be playing in sequence for 5 minutes.
- ➔ Face the „speaker“ (6) to stomach or lower back

SPEAK TO THE BABY

- ➔ Slide the „function mode button“ (8) to „AUX“ position.
- ➔ Plug the provided „microphone“ into the „mic jack hole“ (9).
- ➔ Face the „speaker“ (6) to the stomach or lower back.
- ➔ Use the microphone to talk to your baby.

PLAYING OTHER SOUND SOURCE

- ➔ Slide the „function mode button“ (8) to „AUX“ position.
- ➔ Use the provided „audio cable“ to connect the BH 170 with the other audio source (cassette tape player, MP3, CD Player)
- ➔ Face the „speaker“ (6) to the stomach or lower back.
- ➔ Start playback.

RECORD WOMB SOUND

- ➔ Slide the „function mode“ (8) to „♥“ position.
- ➔ Plug the Headphone into the headphone jack (3) to make sure you can hear the womb sound (The unit must be „ON“)
- ➔ Plug the „audio cable“ to headphone jack (4) to connect the BH 170 and the recorder.
- ➔ Place the „listening cone“ (2) to stomach or lower back
- ➔ Press the „record“ button to start recording the womb sound.

Service hotline

In the case of technical problems, contact our Service hotline. Switzerland: Tel. 0900 00 1675 (national charges, Swisscom at time of going to print: CHF 2.60/min). In the case of claims under the terms of guarantee, contact your sales outlet.

Maintenance

Clean the surface of the housing with a soft, fluff-free cloth. Do not use any cleaning agents or solvents.

Guarantee

SWITEL equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble-free functioning and a long service life. The terms of guarantee do not apply to the batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase. All deficiencies related to material or manufacturing errors within the period of guarantee will be redressed free of charge. Rights to claims under the terms of guarantee are annulled following tampering by the purchaser or third parties. Damage caused as the result of improper handling or operation, normal wear and tear, incorrect positioning or storage, improper connection or installation or Acts of God and other external influences are excluded from the terms of guarantee. In the case of complaints, we reserve the right to repair defective parts, replace them or replace the entire device. Replaced parts or devices become our property. Rights to compensation in the case of damage are excluded where there is no evidence of intent or gross negligence by the manufacturer. If your device does show signs of a defect within the period of guarantee, please contact the sales outlet where you purchased the SWITEL device, producing the purchase receipt as evidence. All claims under the terms of guarantee in accordance with this agreement can only be asserted at the sales outlet. No claims under the terms of guarantee can be asserted after a period of two years from the date of purchase and hand-over of the product.

Disposal

In order to dispose of your device, take it to a collection point provided by your local public waste authorities (e.g. recycling centre). According to the laws on the disposal of electronic and electrical devices, owners are obliged to dispose of old electronic and electrical devices in a separate waste container.
 The adjacent symbol indicates that the device must not be disposed of in normal domestic waste!
 You are legally obliged to dispose of power packs and batteries at the point of sale or in the corresponding containers provided at collection points by local public waste authorities. Disposal is free of charge. The symbols indicate that the batteries must not be disposed of in normal domestic waste and that they must be brought to collection points provided by local public waste authorities. Packaging materials must be disposed of according to local regulations.

Declaration of Conformity

This device fulfils the requirements stipulated in the Directive 1999/5/EC on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity. Conformity with the above mentioned directive is confirmed by the CE mark on the device. To view the complete Declaration of Conformity, please refer to the free download available on our web site www.switel.com.